

INSTRUCTIONS FOR ADJUSTING INSTANT-FIT HEADGEAR

Instant Fit Style Numbers: 140HD, 150H, 170, 130H, 135V, 165

To adjust properly, place headgear on patient's head in the proper position

1) **Figure A** -Draw loose ends of headgear strapping through 3-way adjustment disc or discs (depending on type of headgear).

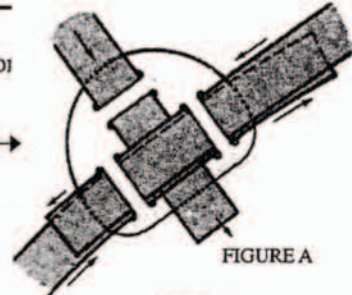


FIGURE A

2) **Figure B**-Some headgear also have a 2-way adjustment loop at back of neck, which can be fitted in the same manner.

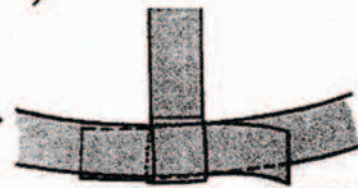


FIGURE B

3) **Figure C**-Apply two (2) staples with stapler pliers.
Note: the crown of the staple should be facing the head with the legs of the staple facing outward injury to wearer. (**Figure D**)
Staple pliers.

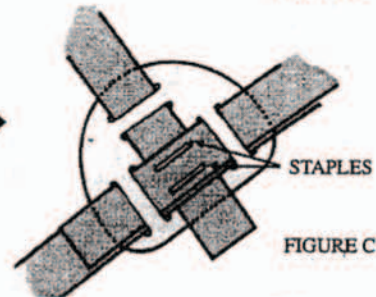


FIGURE C

We are proud to announce that we have improved our #27, disposable chin cap liners, and increased the units in each package so a new liner can be inserted daily to keep the chin cap clean and comfortable throughout treatment. These soft absorbent disposable pads are made of Sontara, a DuPont fabric. Our new liners are designed to extend the life time of Orthoband chin caps when worn daily. The liners simply rest in the inside of the chin cap to protect the chin cap from the inevitable dirt and oils of the skin as patients wear the chin caps over time.

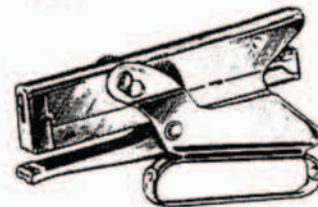


FIGURE D

IMPORTANT NOTICE

SOME PRODUCTS CONTAIN NICKEL COMPONENTS WHICH MAY CAUSE ALLERGIC REACTION. IF THIS OCCURS, CONTACT YOUR DOCTOR IMMEDIATELY.

SOME PRODUCTS CONTAIN NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS IN SOME INDIVIDUALS.

HEADGEAR HAS TEMPORARY FASTENERS ATTACHED FOR SHIPPING PURPOSES. PLEASE REMOVE PLASTIC SNAP PINS BEFORE PERMANENTLY ADJUSTING HEADGEAR.

BARNHART INDUSTRIES, INC.
Anna M. Boehm, President and GM of Barnhart Industries
Founded in 1938

800-325-9973
636-942-3133
636-948-3152 fax